

Tough-PAC® Cleaning instructions

Anti-microbial case

The only case system specifically designed for medical use with inherent proven anti-microbial technology to help with infection control - tested to ISO 22196 standard - shown to kill more than 99.6% of contaminating organisms.

General day-to day cleaning

Day-to-day cleaning should still be carried out as part of any infection or decontamination control policy. These recommendations apply to all Tough-PAC® products. For guidelines and procedures, please contact your local infection control team.

- Wear appropriate PPE
- Turn on iPad using the sleep button in the handle aperture
- Clean all external surfaces of Tough-PAC using a moist general cleaning wipe*
- Dry the surfaces and screen using a clean microfibre cloth

* Follow the user instructions of the cleaning product used.

Cleaning in case of outbreaks or isolation

- Wear appropriate PPE
- Turn on iPad using the sleep button in the handle aperture
- Clean all external surfaces of Tough-PAC using a moist decontamination wipe, air dry*
- Clean again using a detergent wipe particularly the front screen and case contacts (to remove residues)
- Dry the surfaces and screen with a clean microfibre cloth

* 0.1-0.5 % bleach or a peracetic acid wipe. Follow the user instructions of the cleaning product used.