Camera Cable

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取扱説明書
Manual de instrucciones
Instructions for Use
Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso

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English
Before operating the unit, please read this manual thoroughly and retain it for future reference.

For details on how to connect this product to a Sony medical camera, refer to the Instructions for Use supplied with the medical camera.

Note

Do not use this product in conjunction with the PMW-10MD or MCC-3000MT.

Symbols on the product

 **Consult the instructions for use**
Follow the directions in the instructions for use for parts of the unit on which this symbol appears.

 This symbol indicates the manufacturer, and appears next to the manufacturer’s name and address.

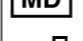
 This symbol indicates the Importer, and appears next to the Importer’s name and registered office address.

 This symbol indicates the European Community representative, and appears next to the European Community representative’s name and address.

 This symbol indicates the UK Responsible Person, and appears next to the UK Responsible Person’s name and address.

 This symbol indicates the Swiss authorized representative, and appears next to the Swiss authorized representative’s name and address.

 This symbol indicates the medical device in the European Community.

 This symbol indicates the date of manufacture.

 **Caution**
For precautions and further information regarding use of this product, refer to the Instructions for Use for the Sony medical camera.

Use with electrosurgical knives and similar devices

If this unit is used together with an electrosurgical knife, etc., the picture may be disturbed, warped or otherwise abnormal as a result of strong radio waves or voltages from the device. This is not a malfunction. When you use this unit simultaneously with a device from which strong radio waves or voltages are emitted, confirm the effect of this before using such devices, and install this unit in a way that minimizes the effect of radio wave interference.

Supplied items

Service Contact List (1)

Information for Customers in Europe (1)

For the customers in the U.S.A.
SONY LIMITED WARRANTY - Please visit <http://www.sony.com/psa/warranty> for important information and complete terms and conditions of Sony’s limited warranty applicable to this product.

For the customers in Canada
SONY LIMITED WARRANTY - Please visit <http://www.sonybiz.ca/pro/lang/en/ca/article/resources-warranty> for important information and complete terms and conditions of Sony’s limited warranty applicable to this product.

For the customers in Europe
Sony Professional Solutions Europe - Standard Warranty and Exceptions on Standard Warranty.
Please visit <https://pro.sony/support-services/primessupport/support-professional-solutions-europe-standard-product-warranty> for important information and complete terms and conditions.

For the customers in Korea
SONY LIMITED WARRANTY - Please visit <http://bpeng.sony.co.kr/handler/BPAS-Start> for important information and complete terms and conditions of Sony’s limited warranty applicable to this product.

Notes

- Always verify that the unit is operating properly before use. SONY WILL NOT BE LIABLE FOR DAMAGES OF ANY KIND INCLUDING, BUT NOT LIMITED TO, COMPENSATION OR REIMBURSEMENT ON ACCOUNT OF THE LOSS OF PRESENT OR PROSPECTIVE PROFITS DUE TO FAILURE OF THIS UNIT, EITHER DURING THE WARRANTY PERIOD OR AFTER EXPIRATION OF THE WARRANTY, OR FOR ANY OTHER REASON WHATSOEVER.
- SONY WILL NOT BE LIABLE FOR CLAIMS OF ANY KIND MADE BY USERS OF THIS UNIT OR MADE BY THIRD PARTIES.
- SONY WILL NOT BE LIABLE FOR THE TERMINATION OR DISCONTINUATION OF ANY SERVICES RELATED TO THIS UNIT THAT MAY RESULT DUE TO CIRCUMSTANCES OF ANY KIND.

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© 2014 Sony Corporation Printed in Japan