Its Cool Wireless

Its Cool Flat Wireless

Its Cool Flat Wireless  
 (Replacement drapes sold separately)

**EN Part functions:** (See figure B.)

- 1 Micro USB port:** Used to connect Battery charging cable. Cover during washing or rinsing.
- 2 Battery charging cable:** Used to charge keyboard battery.
- 3 Port cover:** Used to cover Micro USB port during washing or cleaning.
- 4 Power switch**
- 5 Pairing key**
- 6 LED indicator:**  
 Red: keyboard battery is charging, blinking blue: keyboard is trying to pair, solid blue: keyboard is in paired mode and connected to computer.
- 7 USB Dongle:** Used to transmit keyboard USB signal to computer. Located on bottom of keyboard.

**Installation:** (See figure A.)

Place USB Dongle (7) in computer USB port. Note that each dongle is different and is paired with it's own keyboard. Turn keyboard on (4). Wait for keyboard to pair automatically.

If you are using this keyboard in combination with a Man & Machine wireless C Mouse, you can use one USB dongle to connect both devices. Once you have installed the first one of our devices and let it pair automatically, following the pairing procedure to connect the other device to the same dongle.

**Pairing:** (See figure D.)

The keyboard is already paired with the USB dongle that comes with it. To pair the keyboard with another dongle or to re-pair it with the dongle it comes with: Hold down the blank key (5) on the left side near the

Ctrl & Alt while turning the keyboard on (4). LED indicator (6) will blink blue when pairing. If the USB dongle is plugged in, unplug the USB dongle and plug it back in. Keep the keyboard close to the USB dongle. Once the keyboard is paired and connected to the PC, the blue LED indicator will turn solid blue (6).  
**Note:** The keyboard will not work if it has lost it's pairing. Loss of pairing is signaled by blinking blue light (6). Follow procedure described above to restore pairing.

**Battery charging:**

Use battery charging cable (2). Connect Micro USB connector to keyboard, connect USB connector to 5V USB charging device or powered USB computer port. LED indicator (6) will turn red when charging. The red light will go off when fully charged. Remove and store cable for future use. **Tip:** Turn keyboard off when not in use to maximize battery life. **Note:** The keyboard will not work if the battery is empty. Charge timely following procedure described above.

**Approved cleaning methods keyboard**

- You can wipe the product with a soft cloth that is non-abrasive such as a microfiber cloth, a disinfecting wipe, a soft brush similar to a paint brush in bristle stiffness or a sponge, but not a scourer of any sort.
- You can spray the product with soap and water or an approved disinfectant.
- You can rinse the product with soap and water or an approved disinfectant. Before rinsing, remove cable (2) from port (1) and cover port (1) with cover (3). Make sure cover (3) is fully inserted (See figure C).

**Approved cleaning methods drape**

- You can wipe, spray or rinse the product as described under "Approved cleaning methods keyboard"
- You can immerse the product in soap and water or an approved disinfectant
- You can expose the product to Hydrogen Peroxide Vapour (HPV)
- You can sterilize the product in an autoclave or thermo disinfectant up to 134 °C (270 °F), (tested 200 cycles)

### Approved disinfectants

- Chlorine based, up to 100,000 ppm (10%) bleach content
- Alcohol based
- Glutaraldehyde based
- Phenol based
- Formaldehyde based, up to 370,000 ppm (37%) formaldehyde content
- Oxidizing, such as Hydrogen Peroxide, up to 30,000 ppm (3%) content
- Quaternary Ammonium, up to 20,000 ppm (2%) content

### Restrictions

- Platform: Windows only
- Only clean using the cleaning methods and disinfectants described under "Cleaning instructions" in this manual.
- The product can not be immersed in liquid.
- Always follow the instructions provided by the cleaning product manufacturer.
- Do not use petroleum based cleaning fluids nor Acetone or abrasive cleaning aids and products.
- Do not expose the product to temperatures below 0 °C or above 70 °C (below 32 °F or above 131 °F).
- Do not use temperature or pressure sterilization.

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• Product warranty statement

Man & Machine, Inc. ("Manufacturer") warrants to the original purchaser (the "Customer") that the products it manufactures ("Product") shall be free from defects in materials and workmanship at the time of shipment. Manufacturer's sole obligation under this Warranty shall be limited to repairing or replacing with the same or comparable item, at its option, free of charge, any product returned to Manufacturer freight prepaid that is deemed to have failed under Manufacturer's warranty. Manufacturer shall have no obligation under this Warranty or otherwise if (1) the Product has been damaged by abuse, negligence, accident, mishandling, lightning or flood, or other natural disasters, (2) the Product has been affected by normal wear and tear, (3) the Product has not been operated in accordance with its operating instructions, (4) the Product has been altered or repaired by anyone outside Manufacturer (5) adaptations or accessories have been made or attached to the Product which, in Manufacturer's sole determination, have adversely affected its performance, safety or reliability. If a Product should malfunction or fail during the warranty period, contact the original place of purchase for return options. If purchased directly from Man & Machine, Inc., please submit a Return Material Authorization (RMA). This form can be located on our website at <http://www.man-machine.com/returnsreplacements/>. Please complete requested information and provide a detailed description of the problem. Returned Products must include the RMA number clearly marked on the outside of the package. Manufacturer will not be responsible for any unnecessary items included with any returned Product.

This warranty is the exclusive warranty for any product. Manufacturer specifically disclaims any and all other warranties of any kind, express or implied, including any warranties of fitness for a particular purpose or of merchantability. Manufacturer shall not be liable in tort, including negligence or strict liability, and shall have no liability at all for injury to persons or property. Manufacturer's liability for failure to fulfill its obligation under this limited warranty or any other liability in connection

with a product shall be limited to the amount of the purchase price received by manufacturer for the product. The remedies stated in this limited warranty are the customer's exclusive remedies against manufacturer regarding any product. Under no circumstances shall manufacturer be liable for any indirect, incidental, special or consequential damages, including lost profits and revenues, inconvenience or interruptions in operations, or any other commercial or economic losses of any kind. These limitations and disclaimers are not made by manufacturer where prohibited by law. Some states prohibit limitations of warranties and the customer may have additional rights in those states.

All items returned for warranty repair/replacement must be sent freight prepaid and insured to either the original place of purchase or to the Manufacturer, referencing a valid RMA number:

USA & OTHER COUNTRIES      EUROPE  
**Man & Machine, Inc.**      **Man & Machine Europe**  
**3706 West Street**      **Tappersweg 10c**  
**Landover, MD 20785**      **2031 ET Haarlem**

All warranty repairs or replacements will be returned freight prepaid by ground service to any destination within the Continental United States and Europe. Charges for repairs/replacements shipped back outside this region are the responsibility of the end user.

**Products must be cleaned/sanitized before returning. Returns will not be accepted that have not been disinfected.**

USA & OTHER COUNTRIES      EUROPE  
**1 Year limited warranty**      **1 Year limited warranty**

For current warranty information, visit our website at <http://www.man-machine.com/warranty-statement/>.

The Windows logo is a trademark or registered trademark of Microsoft Corporation in the United States and/or other countries.

USA:  
info@mmimd.com  
+1 301 341 4900

Europe:  
info@mmieu.com  
+31 (0) 23 5450185

UK:  
info@mmieu.com  
+44 (0) 1937 830680