e-medica™

# Computersysteme

Handbuch | Manual

---

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©Baaske Medical GmbH & Co. KG. All rights reserved.

Information in this document has been carefully checked for accuracy; however, no guarantee is given to the correctness of the contents. Photographs and schematic diagrams used in this document are for illustration purposes only. The actual product may vary. This document contains proprietary information protected by copyright. No part of this manual may be reproduced by any mechanical, electronic, or other means, in any form, without prior written permission of Baaske Medical.

This document is subject to change without notice.

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# Manual e-medic™ Computer Systems

# Content

<b>1. Safety instructions</b>	23
Electrical safety and conformity	23
Initial operation	24
Purpose and mode operation	24
Connection of peripheral devices	24
Upgrade and installation of components	25
Action with ingress of foreign objects and liquids	25
Adherence to physical limit values	25
Temperature fluctuation and condensation	26
Choice of location	26
<b>2. Patient environment</b>	27
e-medice™ computer	27
Electrical safety according to EN / IEC 60601-1	27
Safety instructions for operation in the patient environment	28
<b>3. Integration in a medical system</b>	29
<b>4. Operation in a network</b>	29
<b>5. Safe operation</b>	30
Data protection	30
Protection from power outages	30
Care and maintenance	30
Cleaning the housing	31
<b>6. Liability and warranty</b>	32
Contact and Service Hotline	32
Disclaimer	33
Notes	33
<b>7. Connections and controls</b>	34

# Preface

---

## **Dear customer,**

Thank you very much for deciding on a high quality e-medice™ product. Our e-medice™ computer is developed and manufactured in Germany following the strict guidelines of ISO 9001:2015 Quality Management. You have thus bought a quality product with a high manufacturing standard and high life expectancy and have.

We have developed and produced your e-medice™ computer with the best possible components to allow you many years of smooth operation.

Our clear goal: low maintenance, long-term operations so that you can concentrate on your day-to-day business and, in the best case, not notice your computer. It's there, and works.

We wish you a lot of enjoyment with your e-medice™ computer and thank you once again for choosing us as your partner.

## **Your e-medice™ team**

**Note:** To avoid the risk of possible electric shock this unit must only be connected to a mains power supply with an earth conductor protection.

# 1. Safety instructions

**Read carefully prior to initial operation start-up!**

## Notes on Conformity

Your device meets the requirements of electromagnetic compatibility. The e-medice™ 60601-1 computer variants fulfil in particular the electrical safety of EN / IEC 60601-1.

Full Declarations of Conformity are available upon request.

**Hotline +49 (0) 5741 2360270**

## Electrical safety

The computer must only be installed and brought into operation by qualified personnel. The opening of the housing is also only to be carried out by qualified personnel. The e-medice™ computer must only be powered by a 230/240 V mains supply with an attached protective earth conductor. The mains supply must conform to DIN 0100 part 710. The computer is also still connected to the mains supply when it is in a switched off condition. Only through unplugging the computer will it be separated from the mains supply. The operational area must comply with the requirements of EN / IEC 60601-1 or EN / IEC 62368-1. The unit must only be operated using the original mains cable provided. An extension of the power supply connecting cable is not permitted. If the original mains cable has to be replaced with another cable or a longer cable, then only a cable validated by us may be used. With visible damage to a power line this must be replaced immediately with a perfect mains cable that has been validated by us. With transfer or resale of the unit this handbook must be passed on with it.

**Note:** If these instructions are not complied with the unit loses its conformity and there is a danger to life for users, patients and third persons.

### **Initial operation**

When your e-medic™ computer has just been delivered do not under any circumstances connect it straight away! First unpack your new computer very carefully and remove the protective foil. Then leave the computer standing in the room in which it is to be used for at least 2 hours so that the computer can assimilate the room temperature and any possible condensation moisture that has accrued can dissipate. If moisture has built up on the housing wipe this off with a clean, antistatic cloth. As long as there is moisture on the housing the computer must not be connected to the mains supply.

### **Purpose and mode of operation**

This unit is suitable for the storage and processing of data. This unit is developed for long-term use in intermittent operation.

### **Connection of peripheral devices**

Read the operating instructions of the devices to be connected very carefully and follow the installation instructions of the respective products. The cable lengths of the devices connected must not exceed 3 m, otherwise correct operation cannot be guaranteed. Ensure the connectors are seated properly. If you have bought your Pro-Line II computer with an additional graphic card the display connections on the motherboard are disconnected. Connect the monitor to be used to one of the graphics card connectors. These are to be found in the lower area at the back of the computer. With connection to the motherboard sockets for the built in graphic device your monitor will not function.

**WARNING:** With devices that provide the e-medic PCs with additional power via the connecting plug, the supply unit must also be checked according to EN / IEC 60601-1.

## **Upgrade and installation of components**

### **Remove the mains supply plug before opening the housing!**

Any opening of the housing is solely permitted by one of our authorised company's resp. technicians. After opening the e-medic™ 60601-1 computer, and according to applicable legal requirements, a new safety test must be carried out in accordance with EN / IEC 60601-1. For use outside the patient environment, a new safety test according to EN / IEC 62368-1 is required after opening the device.

Before opening the housing it is imperative to remove the mains supply plug and to wait for approx. 1 minute until all electrical current inside the unit has dissipated. Otherwise there will be a danger to life! A subsequent change to the configuration of the unit requires authorisation by e-medic™ Support and only components that have been tested and validated by us may be used. An unauthorised change to the unit configuration will result in an immediate loss of conformity and can lead to a danger to the health of the user and/or the patients and other people as the electrical safety can no longer be guaranteed!

### **Action with the ingress of foreign objects and liquids**

With the inadvertent ingress of foreign objects, liquids or with the development of condensation on the unit it must immediately be separated from the mains supply by removing the plug. Under no circumstances switch the unit on again and contact the in-house technician, resp. your e-medic™ dealer or contact our Support team.

### **Adherence to physical limit values**

In accordance with the technical specifications maintain the specified permissible operating values for temperature, humidity and air pressure without fail as the e-medic™ computer would otherwise be run outside its specifications and safe operation can no longer be guaranteed.

If the limits of the values for operation exceed or are below those stated the unit must be immediately switched off and the mains supply plug removed.

Temperature	+5°C bis +40°C (operating) -10°C bis +60°C (storage)
Humidity	15 % - 85 % (operating, non condensing) 5 % - 90 % (storage, non condensing)
Air pressure	700 hPa - 1075 hPa (operating/ storage)

### **Temperature fluctuation and condensation**

If the unit is exposed to large temperature fluctuations, such as those that can occur through transportation or storage in an unheated location, under no circumstances connect the unit straight away as condensation moisture can accumulate internally. Wipe off visible external condensation with a clean, antistatic cloth and leave it standing without packaging material for at least 2 hours before connecting the e-medice™ computer to the mains supply. The unit can only be connected when no further condensation forms.

### **Choice of location**

The e-medice™ computer must be placed in a dust free, vibration-free and dry location and must not come into direct contact with liquids and combustible gases. Ensure that the computer has a firm, horizontal and level platform. Make sure that the computer has enough room on each side to prevent the build up of heat. Under no circumstances must the rear ventilation grill be covered or blocked as the power supply unit will no longer be sufficiently cooled. If other installation options are chosen it is to be ensured that nobody can be injured due to inadequate fixation of the computer.

## 2. Patient environment

---

To protect your patients from electrical influences avoid simultaneous contact with the patients and the computer housing. The e-medice™ 60601-1 is a so called „Medical Computer“ and was developed and tested for use in the vicinity of patients. It fulfils the requirements of the EN / IEC 62368-1 and EN / IEC 60601-1-2 standards and the power supply fulfils the requirements of EN / IEC 60601-1 standard with respect to the electrical limit values.

The e-medice™ office Computer is only suitable in conjunction with a medical isolating transformer for the patient environment. The output power of the isolation transformer should be at least 150 VA.

### **Electrical safety according to EN / IEC 60601-1**

For the manufacture of a system consisting of computers of the e-medice™ computer series and other electrical devices used in the medical field, the requirements of the valid version of EN / IEC 60601-1 resp. EN / IEC 60601-1-2 are to be taken into account.

Furthermore, the system manufacturer is responsible to market the complete system according to the Regulation (EU) 2017/745 (MDR) on medical devices and, as the case may be, to bring it into operation in the framework of in-house production. Because a computer system without medical software cannot be a medical product as defined by the guidelines of Regulation (EU) 2017/745 on medical devices it is, therefore, a new system made up of medical product and non-medical product.

The e-medice™ computer was certified according to EN / IEC 60601-1-2 and the power supply meets the requirements according to EN / IEC 60601-1.

Depending on the medical purpose of the overall system it may also be necessary to install a so called galvanic separation between an active medical product and a computer system for both as well as for Computers to comply with the requirements of EN / IEC 60601-1. Galvanic separation isolates the wanted signal of the respective PC interface, such as network (Ethernet), USB or DVI interfaces.

The following table shows some of our recommended and tested isolators which can be sourced through your e-medic™ dealer. Data lines that may possibly be connected, e.g. network and ISDN cables, must also be isolated.

Connecting cable type:	Model name:	
USB	STD 1.0 LWL	
Seriell RS-232	RS232 Isolator STD 2.5KV B 9Pin	
LAN-Network	MI 1005 / MI 2005	
BNC/S-VHS	MED Video Isolator 1, 2, 4	
DVI	DVI-Isolator STD1, STD2, STD4	

**Safety instructions for operation in the patient environment**

In general the current valid national laws, standards and guidelines are to be complied with. The e-medic™ computer may be used within the patient environment. In conjunction with active medical products that were introduced according to the Regulation (EU) 2017/745 on medical devices, a new medical electrical system is created that must be re-evaluated. Peripheral devices within the patient environment, such as printers or cameras must also either comply with the requirements of EN / IEC 60601-1 and EN / IEC 60601-1-2 or be additionally equipped with a two-stage protection for the patients. This additional protection can be ensured by the use of isolating transformers or isolators in the connecting cable (see Point 2. „Patient environment“). In non-compliance with the current regulations you are acting with gross negligence and place the lives of your patients and those of uninvolved third parties

in possible danger. If in doubt contact our e-medice™ Support in order to create a safe environment for you and your patients. If you need further support, such as specialist literature or training, you can also contact our e-medice™ Support Team. We will be happy to help you further in an uncomplicated and unbureaucratic manner.

### 3. Integration in a medical system

With the integration of the e-medice™ computers in a medical electrical system the requirements of the valid version EN / IEC 60601-1 and possibly the requirements of the Regulation (EU) 2017/745 on medical devices are to be complied with. The computer must only be electrically connected to and operated with products and devices that comply with the requirements of the standards EN / IEC 60601-1 or EN / IEC 62368-1. Before the integration of the e-medice™ computers in a system, be sure to ask the manufacturer of the medical system if this requirement for the operation with a computer is specified. If operation with a computer is permitted keep strictly to the system manufacturers recommended performance values for the computer unit. Connected data cables must be separately isolated, where appropriate, as well as the connecting cable between the e-medice™ computer and other possible connected devices.

### 4. Operation in a network

The e-medice™ computer may only be operated in a network that complies with the requirements of EN 50173 resp. ISO / IEC 11801. We, as manufacturer of the e-medice™ computers, will not be liable for transmission failures, data loss or electrical surges and short circuits in your network structure that could occur due to non-standard installation.

---

Only allow the installation in your network to be carried out by the network administrator or a person authorised by him. Do not carry out any changes to your network without the authorisation of the network administrator. In addition, in the case of use in the patient vicinity the instructions listed under Point 2. „Patient environment” are to be taken into account.

## 5. Safe operation

### **Data protection**

Through the use of high quality components we have been able to develop a guaranteed durable and stable platform for you. Please make sure, however, that your data is backed up regularly because only you as the user of the computer are responsible for this. We therefore recommend that you back up your data at regular intervals on an external data carrier or, better still, on a computer with two built in hard drives, this run in safe RAID1 array. In accordance with current legislation we, as manufacturer, are not liable for destroyed data and their subsequent costs. Ask your e-medic™ dealer or our e-medic™ support Hotline for an appropriate solution for secure data storage.

### **Protection from power outages**

To avoid data loss occurs with mains supply failures and to ensure safely close down your current application we recommend the use of a medical UPS (Uninterruptible Power Supply). These are available with different output sizes. The VA value given means that with this VA performance decrease the UPS of the connected unit will supply electrical power. Please remember that your monitor should also be connected to the UPS so that the system remains operable in an emergency.

**Care and maintenance**

Clean the unit several times a year. We recommend that you check the e-medice™ filtersystem once a year, before the start of the summer months, for possible dirt build up and, if necessary, to change it. Do not operate the computer for long periods without the anti-bacterial felt air filter as fine dust can enter the housing. In addition to these unit related maintenance recommendations the legal requirements of BGV A 3 (accident prevention regulations for electrical installations and equipment) must be complied with.

**Cleaning the housing**

You can clean the housing and power supply of your e-medice™ computer with a clean, lint-free and anti-static cloth. At all times ensure that the computer is switched off and the mains supply is unplugged before cleaning the computer. If you use liquids for cleaning on no account spray these or pour them directly on the computer or the power supply, instead slightly moisten your cloth with the cleaning liquid. In the process only enough liquid should be in the cleaning cloth so that no drops form when you wipe over the housing. Make sure that no liquid gets into the interior of the unit. As a cleaning agent an all-purpose cleaner, glass and surface cleaner may be used. Desinfection solutions should be based on quaternary ammonium compounds. On no account use acidic, caustic cleaning agents as these could corrode the housing. If liquids have got inside the unit accidentally do not under any circumstances reconnect the unit again but contact your service technician or your e-medice™ dealer.

## 6. Liability and warranty

---

We will grant you a minimum of a 2 year warranty on your e-medic™ computer. (The warranty includes the needed spare parts, the amount of work and the shipping costs back to the customer.)

You only have a right to this voluntary warranty service if your unit has been operated in accordance with the instructions in the handbook and the valid legal requirements.

**In the case of a warranty claim please contact your e-medic™ dealer or our e-medic™ Support Hotline immediately.**

**e-medic™ Support Hotline: +49(0)5741-236027-0**

(Mon.-Thu. from 8.00 AM to 4.45 PM and Fri 8.00 AM to 3.00 PM)

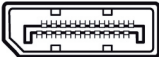
**Support e-mail: [support@e-medic.com](mailto:support@e-medic.com)**

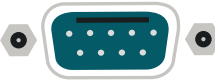

e-medic™ Partner:

# 7. Connections and controls

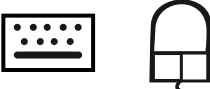Designation	Illustration	Icon
Power supply	Mains connection 	
Digital sound connection	RCA connector (Coax) 	SPDIF-Out 
<b>Sound</b> (external sound sources such as active speakers, microphone) <b>Sound on board</b> If an additional sound card is installed, the sound function on the mainboard is not activated or „Sound onboard“ does not exist	Jack 	Line In  Line Out  Mic 
Connection for USB devices	USB Port 	USB 
Network or DSL modem	RJ-45 Jack 	Netzwerk  LAN 
Connection for digital video signal (monitor)	DVI-plug 	DVI 
Audio/video interface (monitor)	PisplayPort 	DisplayPort DP1 DP2

Designation	Abbildung	Symbol
Digital Audio/Video Interface (Monitor)	HDMI-Plug 	HDMI
Serial interface	Com-Plug 	COM 
PS2 Keyboard / Mouse plug	PS2-Plug 	Keyboard    Mouse 
main switch		Switch on / off. The device can not be started from the network (Wake on LAN) when it is switched off. No power is consumed.
On / off switch		Switch to start up the system.

Connections depending on the board configuration.

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Baaske Medical GmbH & Co. KG • 32312 Lübbecke • [www.e-medical.de](http://www.e-medical.de)
**e-medical™** Die Marke der Baaske Medical GmbH & Co. KG